
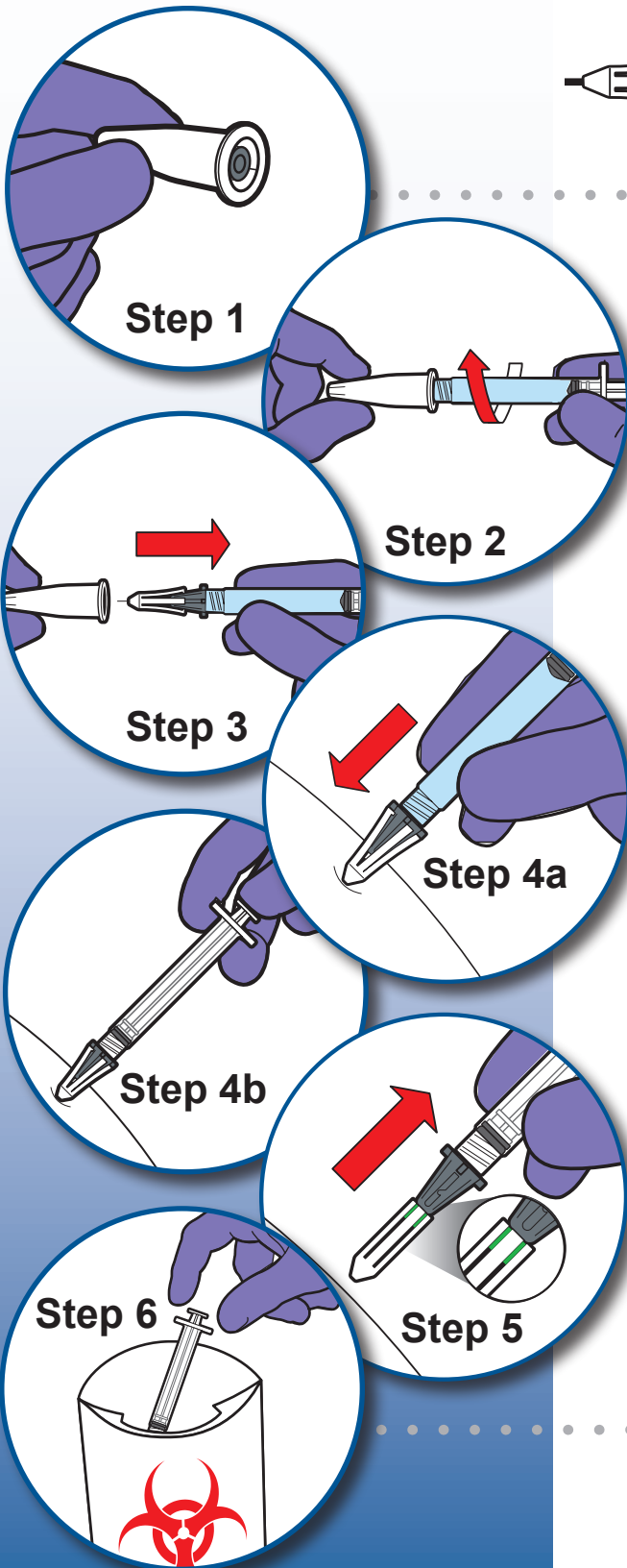


# Instructions for Use

The NovaGuard safety needle is to be used for subcutaneous injections (i.e. syringes up to and including 3mL).

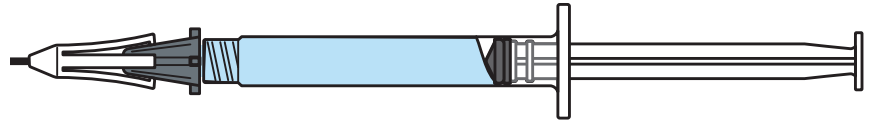

## Step 1:

Before handling the components, wash your hands with soap and hot water. Put on gloves prior to attaching the NovaGuard to the filled syringe and giving the injection.

Remove the device from the sterile pouch by peeling open the pouch. Be careful not to touch the open end of the sterile device inside the pouch. Discard the pouch.

## Step 2:

Hold the open package firmly, with the opening away from your hand.

Attach a filled syringe to the luer connector of the device by inserting the syringe into the package and turning the syringe in a clockwise direction using gentle pressure. Continue to turn until the syringe advances into the tube and can no longer be turned.

## Step 3:

While maintaining clockwise pressure, pull the syringe out of the package and discard the packaging. Take care not to touch the exposed end of the needle. The syringe is now ready for administration.

Once removed from the package, the safety needle should be used immediately.

Note: If resistance is felt upon removing the safety needle from the package, STOP, turn the syringe clockwise again until it can no longer be turned and while maintaining clockwise pressure pull the syringe out of the package.

## Step 4:

The syringe is now ready to use. Inject the drug following standard procedures.

## Step 5:

As the syringe is withdrawn, the device will automatically cover the needle. Exercise care when handling the used syringe because all needles have the potential to retain residual fluid.

Visible green bars indicate that the NovaGuard safety needle is in the locked position.

## Step 6:

Dispose of the syringe in an approved sharps container.

### RX Only.

Single use only. Contents are sterile unless package is opened or damaged. West™ and the diamond logo and NovaGuard™ are trademarks of West Pharmaceutical Services, Inc. in the United States and other jurisdictions.