

MixJect[®]

Before handling the components, wash your hands with soap and hot water.

Place the package on a clean, flat surface.

Remove the Flip-Off[®] button from the drug product vial.

Disinfect the vial injection site with an alcohol pad.

Step 1:

Peel the cover off the MixJect pack.
Do not remove the device from the pack.

Place the MixJect device on top of the vial, using the blister pack as a holder.

Step 2:

Seat the device on the vial by pushing down until the spike penetrates the elastomeric stopper and the device snaps in place.

Remove the blister pack and discard it.

Step 3:

Connect the diluent prefilled syringe to the MixJect.

Step 4:

Transfer the diluent to the vial by slowly pushing the plunger in all the way.

Gently swirl the vial to make sure the product is thoroughly mixed.

Step 5:


Invert the vial and slowly pull the plunger back to withdraw the mixed drug from the vial into the syringe.

Step 6:

Remove the vial from the MixJect assembly.

The drug is now ready for injection.

Follow normal safety practices to administer the drug.

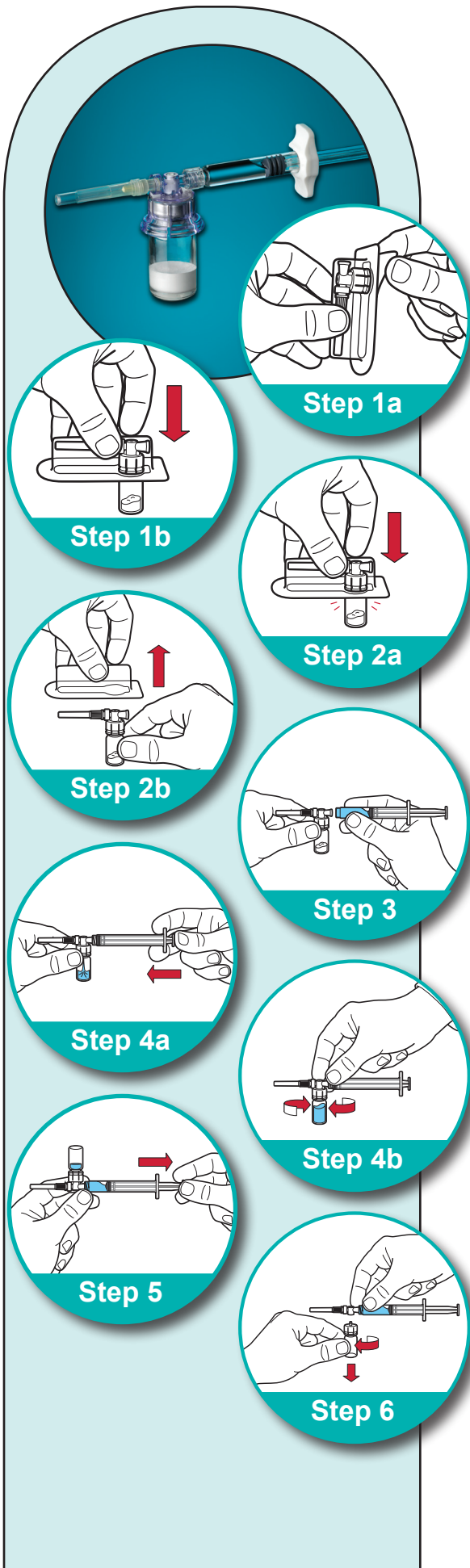

West 

MIP
MEDIMOP Medical Projects Ltd.
A West Company

westReconstitution.com

For detailed directions on reconstituting drugs with the MixJect device,
please refer to the device's Instructions For Use.

Flip-Off[®] is a registered trademark of West Pharmaceutical Services, Inc. in the United States and other jurisdictions.
The MixJect[®] system is the subject of U.S. Patent no. 6,238,372 B1.
MixJect[®] is a registered trademark of Medimop Medical Projects Ltd.
Copyright ©2010 West Pharmaceutical Services, Inc. 5514b MixJect