

# MixJect<sup>®</sup>


## Instructions for Use

Before handling the components, wash your hands with soap and hot water.

Place the package on a clean, flat surface.

Remove the Flip-Off<sup>®</sup> button from the drug product vial.

Disinfect the vial injection site with an alcohol pad.

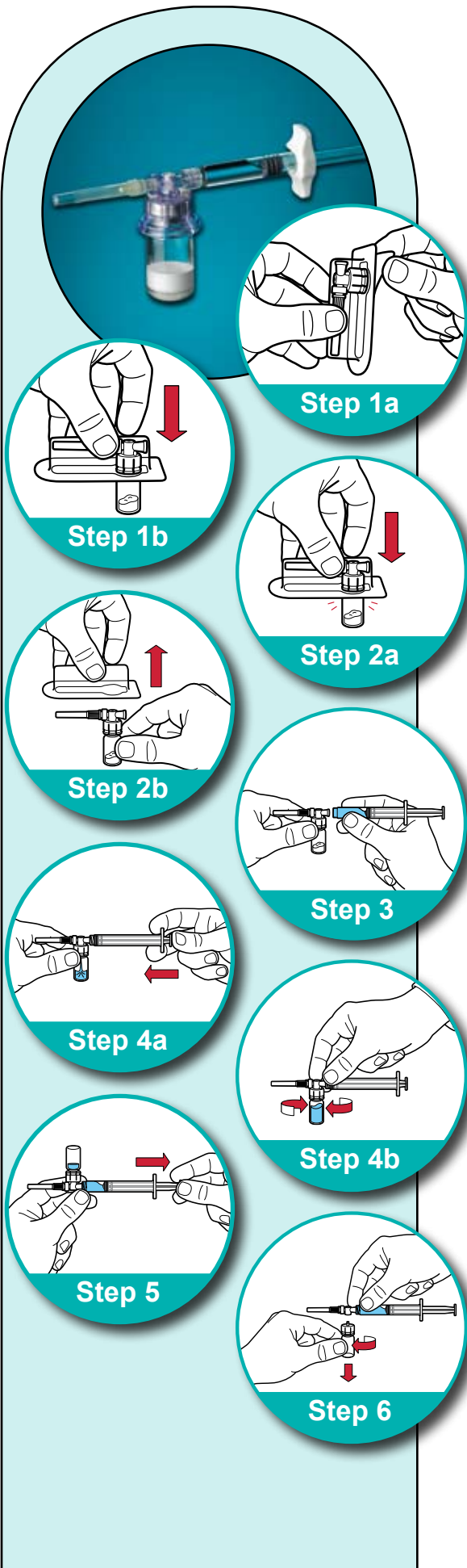

### Step 1:

Peel the cover off the MixJect pack.  
Do not remove the device from the pack.

Place the MixJect device on top of the vial, using the blister pack as a holder.

### Step 2:

Seat the device on the vial by pushing down until the spike penetrates the rubber stopper and the device snaps in place.

Remove the blister pack and discard it.

### Step 3:

Connect the diluent prefilled syringe to the MixJect.

### Step 4:

Transfer the diluent to the vial by slowly pushing the plunger all the way in.

Gently swirl the vial to make sure the product is thoroughly mixed.

### Step 5:

Invert the vial and slowly pull the plunger back to withdraw the mixed drug from the vial into the syringe.

### Step 6:

Remove the vial from the MixJect assembly.

The drug is now ready for injection.

Follow normal safety practices to administer the drug.



[westReconstitution.com](http://westReconstitution.com)

Flip-Off<sup>®</sup> is a registered trademark of West Pharmaceutical Services, Inc., in the United States and other jurisdictions.  
The MixJect<sup>®</sup> system is the subject of U.S. Patent no. 6,238,372 B1.  
MixJect<sup>®</sup> is a registered trademark of Medimop Medical Projects Ltd. 4573b MixJect